


Emergency Disconnect Procedure

Model AK 98
For Paramedics
Catheter

1

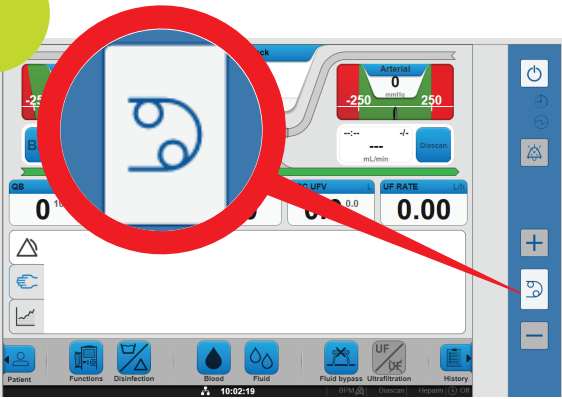


PRESS BLOOD PUMP BUTTON.
Blood pump will stop and
button will start flashing.

2




**USING THE PRODUCTS
FOUND IN THE PATIENT'S
EMERGENCY KIT...**

3

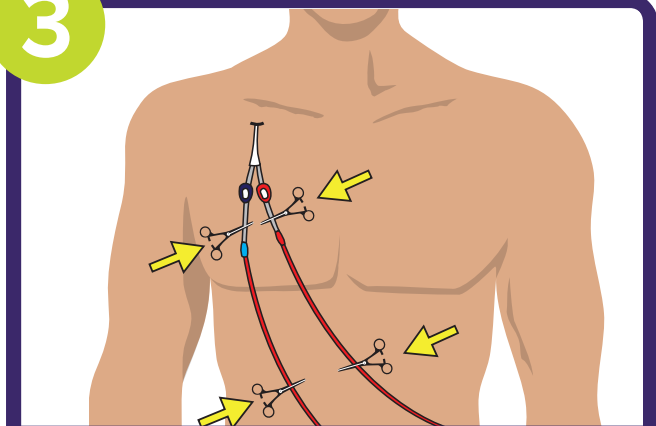

**CLOSE CLAMPS ON THE
CATHETER AND BLOODLINES.
ADD SAFETY CLAMPS BELOW
AND ABOVE THE CONNECTION.**

4

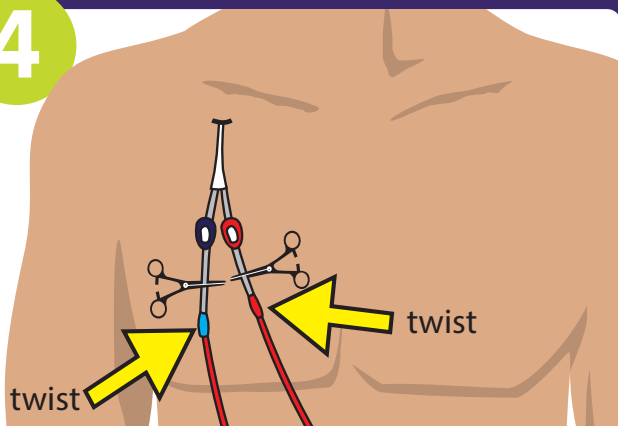

**DISCONNECT LINES AT THE
CONNECTION SITE
BY TWISTING.**

5



**TRANSPORT THE PATIENT
AS-IS TO HOSPITAL.**

Produced by **BCRenal**
Provincial Health Services Authority

