


# Emergency Disconnect Procedure For Paramedics

for the AK 95S

**CATHETER**

1

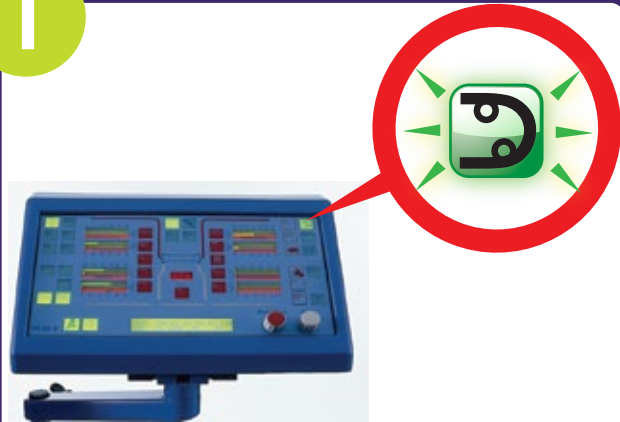

## PRESS BLOOD PUMP BUTTON


Blood pump will stop and button will start flashing.

2




USING THE PRODUCTS FOUND IN THE PATIENT'S CLAMP & CUT KIT...

3

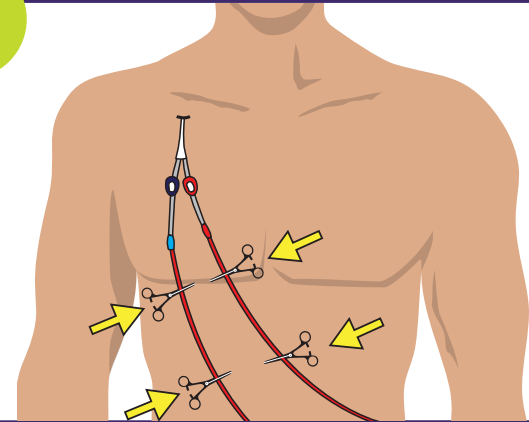

CLOSE TWO CLAMPS ON EACH OF THE TWO BLOODLINES

4

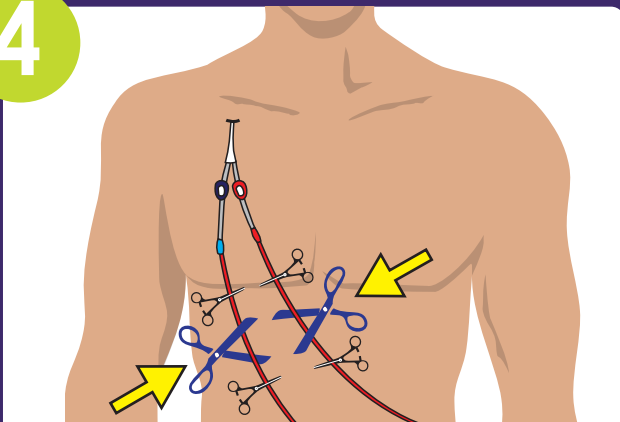

CUT BETWEEN THE CLAMPS

5



TRANSPORT THE PATIENT AS IS TO HOSPITAL

Produced by **BCRenal**  
an agency of the Provincial Health Services Authority

