


# Emergency Disconnect Procedure For Paramedics

for the AK 95S

**GRAFT or FISTULA**

1

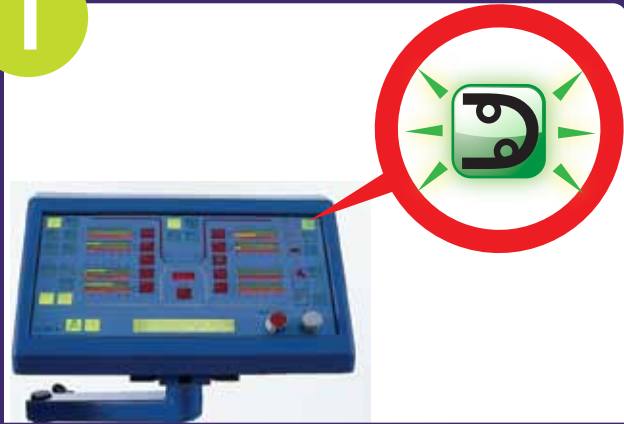

## PRESS BLOOD PUMP BUTTON


Blood pump will stop and button will start flashing.

2




USING THE PRODUCTS FOUND IN THE PATIENT'S CLAMP & CUT KIT...

3

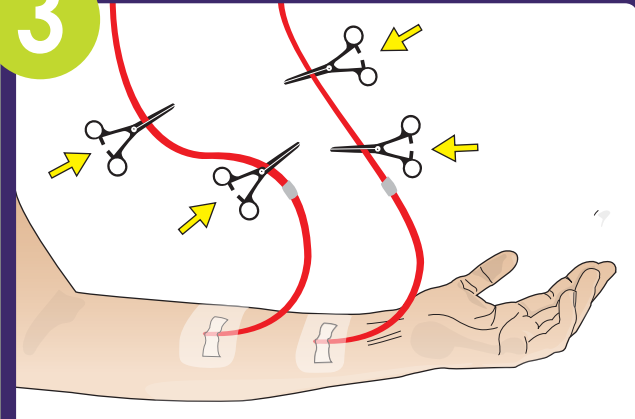

CLOSE TWO CLAMPS ON EACH OF THE TWO BLOODLINES

4

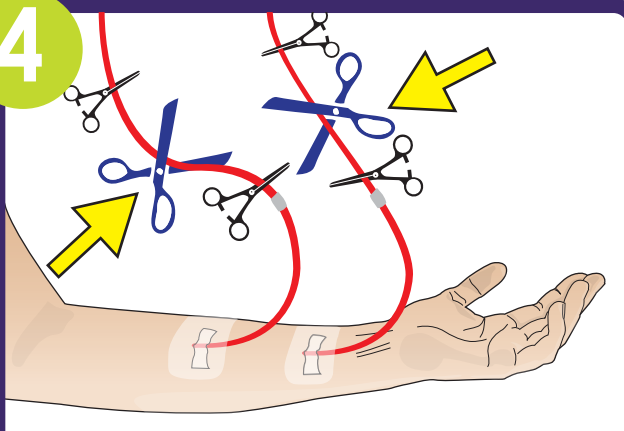

CUT BETWEEN THE CLAMPS

5



TRANSPORT THE PATIENT AS IS TO HOSPITAL

Produced by **BCRenal**  
an agency of the Provincial Health Services Authority

**fraserhealth**  
Better health. Best in health care.

**Interior Health**

**northern health**  
the northern way of caring

**Providence HEALTH CARE**  
How you want to be treated.

**Vancouver Coastal Health**  
Promoting wellness. Ensuring care.

**VANCOUVER ISLAND health authority**

**Provincial Health Services Authority**  
Province-wide solutions. Better health.

**BRITISH COLUMBIA**