


## **SCHEDULED Carbon Filter Rinsing Procedure for the WRO 95 (with Integrated Disinfection)**

This procedure is to be followed after the replacement of any or all 10" or 20" carbon filters. After the initial flushing of new filters at S3 in the above procedure:

1. Press the ON/OFF Switch to turn on the AK 95S. Wait 30 seconds for the lamp test on the AK 95 to complete. A green rectangular button with a white horizontal line. The word 'ON' is in green above the line, and 'OFF' is in green below the line.
2. Press Rinse/Drain button once – “Rinse 10 minutes” is flashing in the display window
3. Turn set knob to have 1:00 flashing in the display
4. Press and Hold Rinse/Drain button for 3 seconds until it lights solid.
5. Allow the AK95S to rinse for an hour. Silence and ignore all alarms on the WRO95.
6. The AK95S will turn off after the rinse is completed. Silence any alarms on the WRO95.
7. Perform an Integrated Dialox Procedure as per instructions in this guide.

---

**Siege social/Head Office:**

9157, rue du Champ-d'Eau, St. Léonard (Québec) H1P 3M3 • Tél. (514) 327-1635 • Fax: (514) 327-0822 • 1-800-361-4098

**Sales Office/Succursale:**

2 East Beaver Creek Road, #4, Richmond Hill, (Ontario) L4B 2N3 • Tel: (905) 762-0690 • Fax: (905) 762-0685 • 1-800-387-5277